Cannula needles with olive / Aiguilles boutonnées / Knopfkannüle / Aghi bottonuti / Kaniula z oliwką / Kanylové jehly s olivkou

REF		REF		REF		REF	
8001726	60mm, 19G	8001760	100mm, 16G ⁽¹⁾	8001788	80mm, 21G	8001843	120mm, 15G
8001727	100mm, 18G	8001761	80mm, 18G	8001789	100mm, 19G	8001844	120mm, 17G
8001736	100mm, 18G 	8001762	40mm, 17G	8001798	100mm, 13G ⁽¹⁾	8001845	55mm, 17G ⁽¹⁾
8001738	80mm, 17G	8001763	35mm, 21G	8001841	120mm, 13G	8001890	80mm, 18G ⁽¹⁾
8001752	80mm, 14.5G	8001764	80mm, 18G	8001842	120mm, 14G	8001988	150mm, 18G

⁽¹⁾ EN Curved, FR Courbé,
DE Gekrümmt, IT Piegato,
PL křivka, CS Zahnuté

EN - Instruction for Use

Intended use

Single use, metal instruments used for cleaning, clearing and rinsing of the wound during general surgery.

Indication for use

Irrigation cannulas are indicated for use when delivery of fluids is needed to irrigate, clean, or rinse wounds during medical or surgical procedures.

Target Population

Adults, pediatrics and neonates.

Intended users and use environment

The devices are to be used in a healthcare facility by professional healthcare practitioners trained to conduct general surgeries.

Contraindications and Side-effects

Devices not expected to have contraindications or side-effects.

Directions for use

- Read instructions and check expiry date before use.
- Visually inspect the integrity of the product and packaging before use.
- Carefully check the olive is well attached to the cannula before use.
- Use the product immediately after opening the individual blister packaging.

- Connect the base with other medical devices to use the cannula needle (for example: syringe, connector, etc.)
- Flush before use to ensure absence of particulate; do not use if particulates are observed.
- After using the cannula needle, check if the olive is still attached to the cannula.
- Dispose immediately after use in accordance with local healthcare waste regulations (device contaminated with potentially infectious substances of human origin).

Warning/Precaution

- Do not bend the cannula.
- Do not use the device as a rubbing utensil against tissues or bones.
- Make sure other mechanical devices do not come into contact with the cannula.
- Do not use if the product / packaging is tampered or damaged
- Single use device: Do not re-use (risk of infection).
- Do not attempt to clean, disinfect, or sterilize for reuse
- Do not use within the central nervous system or the central circulatory system

Complementary information:

- Non active, non-implantable and surgically invasive device for transient use (<60min), no measuring function.
- No specific storage requirements.

Users should report any serious incident related to the device to the Manufacturer (vigilance@applimed.ch) and the National Competent Authority.

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